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© WHITEsmile GmbH · #0125 · Version 3 · 03/2009

GB Instructions for WHITEsmile
POWER BLEACHING Gel XTRA

Introduction

WHITEsmile POWER BLEACHING Gel XTRA is a dual syringe-system to whiten the teeth in the dental office quickly and easily – normally in a single session.

WHITEsmile POWER BLEACHING Gel XTRA is available in 2 variations: 38 % Hydrogen Peroxide (H₂O₂) recommended mainly for bleaching without lamp. LIGHT (32% HP) for the use with bleaching lamps or laser. (Especially recommended for use with the WHITEsmile Bleaching Lamp).

The use of both variations is exactly the same other than using a bleaching lamp during working time of the gel or not. The gel is highly viscous to prevent dripping. It can be applied easily to the teeth using the mixing tips. The mixed gel is red. The used syringe can be re-closed allowing the remaining gel to be used for the next patient, with a new mixing tip.

Warning

Because of the high concentration of Hydrogen Peroxide WHITEsmile POWER BLEACHING Gel XTRA is highly reactive. To avoid contact with skin or eyes we recommend the use of gloves and safety goggles. If the gel does come into contact with skin, remove it and rinse immediately with water. White spots may appear in the affected area and you may experience a burning sensation. The symptoms should disappear after approx. 30 – 60 minutes. If the gel comes into contact with the eyes, rinse immediately and consult a doctor. Any dental work (crowns, fillings etc.) should be done 1 – 2 weeks after the treatment in order to match the colour of the newly whitened teeth. Please note that the adhesive strength of any dental work may be lessened in the 2 – 3 days following bleaching treatment. Do not perform whitening treatments under anaesthetic. WHITEsmile does not recommend the use of POWER BLEACHING gel on patients who are pregnant, lactating or under the age of 16.

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1) Preparation

After examining the patient the current tooth colour should be documented. It may be advisable to begin the treatment with a professional teeth cleaning. The teeth should be kept free by using a lip holder, lip retractor or similar instrument. If necessary, use cotton wool balls and apply Vaseline to the patient's lips.

2) Application of WHITEsmile GINGIVA PROTECTOR

Attach the dispensing tip to the top of the syringe of the WHITEsmile GINGIVA PROTECTOR.

Dry the gingiva carefully to ensure that the WHITEsmile GINGIVA PROTECTOR will hold and seal during the complete treatment. Place the WHITEsmile GINGIVA PROTECTOR directly on the gingiva and cover 3 – 4 mm of the gingiva and 0.5 – 1 mm of the teeth. Now light-cure the WHITEsmile GINGIVA PROTECTOR with a UV-lamp for 30 – 40 seconds in an arching movement over the material. Instead of the WHITEsmile GINGIVA PROTECTOR a rubber dam can be used.

Tip:

The gingiva protector can also be light cured with the WHITEsmile BLEACHING LAMP!

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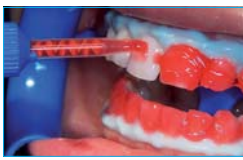
3) Whitening procedure

Attach the mixing tip to the top of the dual-syringe of the *WHITEsmile* POWER BLEACHING Gel XTRA. Apply the gel directly to the surface of the teeth (to whiten single teeth, *WHITEsmile* GINGIVA PROTECTOR and the *WHITEsmile* POWER BLEACHING Gel XTRA can also be applied lingual). The gel should be applied about 1 – 2 mm thick (avoid contact with other parts of the body and clothes).

Allow the gel to work about 12 – 15 minutes. Reduce the application time and cycles if patient experiences sensitivity. *WHITEsmile* POWER BLEACHING Gel XTRA can also be activated with a curing lamp (10 seconds for each tooth is enough). Special bleaching lamps or lasers can also be used (check with the manufacturers for details about this kind of usage). The material breaks down faster, so the treatment time can be reduced to 10 minutes.

Remove the gel from the teeth using suction – do not rinse – then apply a fresh layer of gel to the teeth. This process can be repeated three times.

After the last treatment remove the gel first using suction and then by rinsing.

Now the GINGIVA PROTECTOR, lip holder and cotton wool balls can be removed. Finally the patient should rinse their mouth with warm water.

When bleaching lamp is used:

Place bleaching lamp close to the patient's mouth during the gel application and turn the lamp. In case of the *WHITEsmile* Bleaching lamp only eye protection with safety goggles for patient/operator are mandatory. For other bleaching lamps follow instruction of the lamp manufacturer.

After less than one hour your patient will be able to enjoy his new white smile – thanks to your treatment.

The tooth colour after treatment should also be documented for your records. After the treatment apply a fluoride varnish to the teeth. You can finish off the treatment and lower the risk of unpleasant sensitivity for the patient with *WHITEsmile* AFTER BLEACHING MOUSSE – apply to the teeth and leave for about 10 minutes. Recommend *WHITEsmile* WHITENING TOOTHPASTE with 30 % Xylitol to your patient to maintain results and protect the teeth.

Side effects

If the patient feels a burning sensation in the mouth, treatment should be stopped at once and the gel should be removed. Sensitivity can be reduced by using a fluoride gel or *WHITEsmile* AFTER BLEACHING MOUSSE after the treatment. For safety reasons, the patient should also wear safety goggles. For further information please refer to the general *WHITEsmile* instructions for dentists.

Storage and shelf life

WHITEsmile POWER BLEACHING Gel XTRA 38% should be stored in a cool place or refrigerator (4 – 12 °C). The expiry date on the package or syringe refers only to products that have been stored correctly. Do not freeze the gel and protect it from heat and sunlight.

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