

WHITEsmile® – the world of tooth whitening

Method	Application	Application Time per Session	Number / Frequency of Applications
 <p>WHITEsmile® HOME BLEACHING with laboratory custom-made trays</p> <p>WHITEsmile® Tooth Whitening Gel is a professional tooth whitening treatment to be used at home by the patient with laboratory custom-made trays. Due to the neutral pH-value of the mint flavored gel and the intensive care with WHITEsmile® AFTER BLEACHING MOUSSE this treatment is especially comfortable for the patient and gentle to the teeth. For the dental clinic this product has the advantage of minimized chair-time (impression taking only). To the patient it offers the possibility to repeat the treatment as many times as desired.</p>	 	overnight, 4 hours	5–12
		overnight or day-time, 3 hours	5–12
	 	daytime, 2 hours or 2 × 45 mins	5–12
		daytime, 30–60 mins	5–10
 <p>WHITEsmile® POWER BLEACHING chairside</p> <p>WHITEsmile® POWER BLEACHING XTRA 38% is the fast method of chairside tooth whitening in the dental clinic.</p>		3×15 mins (max. 4×15 mins)	1 session, in exceptional cases 2 sessions
 <p>WHITEsmile® POWER BLEACHING chairside with bleaching lamps</p> <p>WHITEsmile® POWER BLEACHING XTRA LIGHT with bleaching lamp is the most effective, fastest and most attractive method of chairside tooth whitening.</p>		3×15 mins (max. 4×15 mins)	1 session, in exceptional cases 2 sessions
 <p>WHITEsmile® Aftercare</p> <p>WHITEsmile® AFTER BLEACHING MOUSSE provides the teeth with everything they need after a tooth whitening treatment.</p>		10–15 mins after bleaching	as required
 <p>WHITEsmile® Daily Care</p> <p>WHITEsmile® WHITENING TOOTHPASTE is the ideal care for white teeth. Suitable for daily use.</p>		2 mins	2 × daily, mornings and evenings